# v VARDIS

SWITZERLAND

BIOMIMETIC DENTAL SCIENCE

## CURODONT™ REPAIR

Guided Enamel Regeneration for the treatment of initial caries

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CURODONT™ REPAIR from vVARDIS is the first biomimetic system for the treatment of initial caries by enamel regeneration:

- Patented, clinically proven *Monomer-Peptide 10* technology
- Initiates enamel regeneration at the depth of the lesion
- Between 86% to 100% inactivation and regression of initial carious lesions
- Non-invasive, painless and easy to use
- Suitable for all patients, including children

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professional.vardis.com

## APPLICATION (5-8 MINUTES)

Treatment by dental hygienist or dentist

- Professional tooth cleaning
- Cleaning of the affected tooth surface with 2% sodium hypochlorite for 20 sec.
- Application of 35-37% orthophosphoric acid to the area in question for 20 sec.
- Rinse and dry gently
- Application of CURODONT™ REPAIR

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
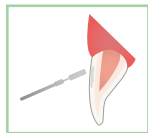
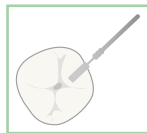
Occlusal Caries

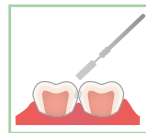Interproximal Caries

1 applicator of CURODONT™ REPAIR can treat up to 1-3 initial carious lesions

- Initial approximal caries D1, D2, (D3, non-cavitated)
- Initial occlusal caries
- Initial smooth surface caries
- Initial caries in deciduous teeth

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## PRECAUTIONS

CURODONT™ REPAIR should only be used by dental professionals

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